

### Mask Instructions



Open bag and remove a mask, put the end with the nosepiece up, with the light blue side facing outward, and gently pull the ear loops to hang the mask on both ears.



Gently press the nosepiece to fit it to the bridge of the nose, then gently press and hold the nosepiece to pull the lower end of the mask down to the jaw, so that the folds are fully extended.



Position the mask so that it covers the user's nose, mouth, and jaw; ensuring the tightness of the mask.

- 1. Keep away from rain
- 2. Keep away from sunlight
- 3. Do not reuse

### Notes

1. This product is not intended for medical use.
2. Product is not intended for use on children.
3. Product is single use and recommended wearable life is 4- to 6 hours.
4. Use product prior to expiration date.
5. For individuals who are allergic to nonwoven materials, please use caution when using the product. If there is any adverse reaction while wearing, please discontinue use of the product.
6. Product is not washable and should be stored in a dry location. Keep away from fire.



Executive Standard: GB/T32610-2016  
Manufacturer: VIOLET HOME TEXTILE TECHNOLOGY CO., LTD  
Address: Zhongliang Crossroad, 336  
Provincial Highway, Zhamao Village  
Town, Yongzhou District, Nanjing,  
Jiangsu, China

PRODUCTION DATE: 6/28/2020  
EXPIRATION DATE: 6/27/2022  
LOT NO: 20200628 7457



MADE IN CHINA

PENGAD 800-831-6968  
PLAINTIFF'S  
EXHIBIT

12